## **MEEASY™ BTE 312, 13 & 675**

with Audiotone® Pro Technology User Manual

# Content

<b>Welcome</b>	<b>4</b>
<hr/>	
<b>Your hearing aids</b>	<b>5</b>
<hr/>	
Hearing aid type	5
Getting to know your hearing aids	5
Components and names	6
Controls	9
Settings	11
<b>Batteries</b>	<b>12</b>
<hr/>	
Battery size and handling tips	12
Replacing batteries	13
<b>Daily use</b>	<b>14</b>
<hr/>	
Turning on and off	14
Inserting and removing the hearing aids	15
Adjusting the volume	18
Changing the hearing program	18
Locking the controls (optional)	19
<b>Special listening situations</b>	<b>20</b>
<hr/>	
On the phone	20
Audio induction loops	21
Audio input (FM system)	22

<b>Maintenance and care</b>	<b>23</b>
Hearing aids	23
Ear pieces and tubes	24
Professional maintenance	28
<b>Further information</b>	<b>29</b>
Safety information	29
Symbols used in this document	29
Troubleshooting	29
Country-specific information	30
Service and warranty	31

# Welcome

Thank you for choosing our hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This manual, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

## **CAUTION**

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

# Your hearing aids

This user manual describes optional features that your hearing aids may or may not have.

Ask your Hearing Care Specialist to indicate the features that are valid for your hearing aids.

## Instrument type

Your hearing aids are BTE (Behind-The-Ear) models. A tube delivers the sound from the hearing aid into the ear. The hearing aids are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.


## Getting to know your hearing aids

It is recommended to familiarize yourself with your new hearing aids. With the hearing aids in your hand try using the controls and note their location on the hearing aid. This will make it easier for you to feel and press the controls while wearing the hearing aids.

## Components and names

This user manual describes several types of hearing aids. Use the following pictures to identify the type of hearing aid that you use.

### MEEASY™ BTE 312

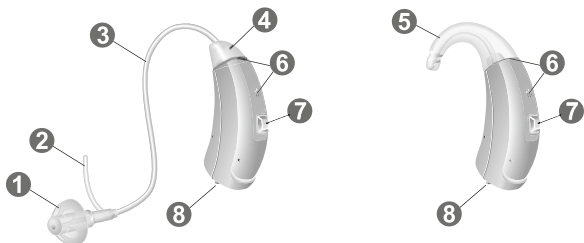- 
- |                               |                                       |
|-------------------------------|---------------------------------------|
| ① Ear piece (OpenTip)         | ⑤ Earhook                             |
| ② Retention strand (optional) | ⑥ Microphone openings                 |
| ③ Tube (ThinTube)             | ⑦ Push button (control)               |
| ④ Adapter for ThinTube        | ⑧ Battery compartment (on/off switch) |
-

## MEEASY™ BTE 13

## MEEASY™ BTE 675

---

**1** Ear piece

**2** Adapter for ear piece

**3** Tube for earhook

**4** Earhook

**5** Microphone openings

**6** Push button (control)

**7** Rocker switch (control)

**8** Battery compartment  
(on/off switch)

---

## Ear pieces and tubes

### Ear pieces

### Size

Standard ear pieces

 OpenTip open or closed

 OpenTip semi-open

 OpenTip double

Custom-made ear pieces

Examples:

### Tubes

### Length

Standard tube (ThinTube)

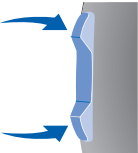
Tube for earhook

You can easily exchange the standard ear pieces and clean the standard tube. Read more in section "Maintenance and care."

## Controls

With the controls you can, for example, adjust the volume or switch hearing programs. Your hearing aids either have a push button and a rocker switch or a push button only.

Your Hearing Care Specialist has programmed your desired functions to the controls.

Control	Left	Right
Push button	<input type="radio"/>	<input type="radio"/>
		
Rocker switch	<input type="radio"/>	<input type="radio"/>
		

<b>Rocker switch function</b>	<b>L</b>	<b>R</b>
Press briefly:		
Volume up/down	○	○
Sound mixer (audio input)	○	○

L = Left, R = Right

<b>Push button function</b>	<b>L</b>	<b>R</b>
Press briefly:		
Program change	○	○
Press long:		
Standby/turn on	○	○

L = Left, R = Right

<b>Push button and rocker switch function</b>	<b>L</b>	<b>R</b>
Lock/Unlock the controls	○	○
Read more in section "Locking the controls."		

L = Left, R = Right

## Settings

### Hearing programs

---

1

---

2

---

3

---

4

---

Read more in section "Changing the hearing program."

### Features

- Power-on delay enables whistle-free insertion of hearing aids.

Read more in section "Turning on and off."

- A telecoil is built into the hearing aid, so it can connect with audio induction loops.

Read more in section "Audio induction loops."

# Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

## Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.

---

Battery size:                      ○ 312      ○ 13      ○ 675

---


- Always use the correct battery size for your hearing aids.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

## Replacing batteries

### Removing the battery:

- ▶ Open the battery compartment.

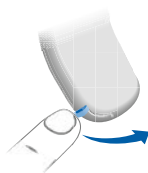- ▶ Push the battery out (refer to the picture).

### Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.

- ▶ Insert the battery with the "+" symbol facing up (refer to the picture).

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly.

Do not attempt to close the battery compartment by force. It can be damaged.

# Daily use

## Turning on and off

You have the following options to turn your hearing aids on or off.

### Via battery compartment:

- ▶ Turning off: Open the battery compartment fully.

- ▶ Turning on: Close the battery compartment.

### Via push button:

- ▶ Turning on or off: Press and hold the push button for several seconds. Refer to section "Controls" for the settings of your controls.

After turning on, the previously used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when an hearing aid is being turned on or off.

When the power-on delay is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

## Inserting and removing the hearing aids

Your hearing aids have been fine-tuned for your right ear and left ear. Colored markers indicate the side:

- red marker = right ear
- blue marker = left ear

MEEASY™ 3S


MEEASY™ 3P, 3SP

### Inserting a hearing aid:

- ▶ Hold the tube close to the ear piece.
- ▶ Carefully push the ear piece into the ear canal **1**.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.

- ▶ Lift the hearing aid and slide it over the top of your ear **2**.

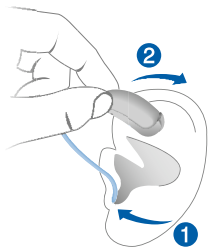## CAUTION

Risk of injury!

- ▶ Insert the ear piece carefully and not too deeply into the ear.

**i**

- It may be helpful to insert the **right** hearing aid with the **right** hand and the **left** hearing aid with the **left** hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


**The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:**

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

## Removing a hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ①.
- ▶ Hold the tube close to the ear piece and pull the ear piece out carefully ②.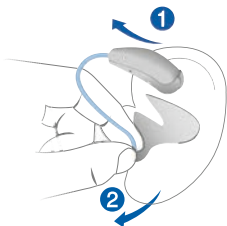

### CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care."

## **Adjusting the volume**

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the rocker switch briefly.

Refer to section "Controls" for the setting of the rocker switch.

An optional signal tone can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

## **Changing the hearing program**

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the push button briefly.

Refer to section "Controls" for the setting of your controls. Refer to section "Settings" for a list of your hearing programs.

## Locking the controls (optional)

To prevent any accidental operation you can lock the controls. When the controls are locked, both push button and rocker switch are disabled.


- ▶ To **lock** the controls, press the **top** of the rocker switch and hold it. Simultaneously press the push button on the same hearing aid for 3 seconds.
- ▶ To **unlock** the controls, press the **bottom** of the rocker switch and hold it. Simultaneously press the push button on the same hearing aid for 3 seconds.

Whether this function is configured for your hearing aids is listed in section "Controls."

## Special listening situations

### On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

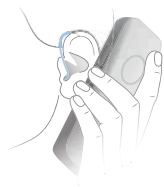### Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Settings."

## Audio induction loops

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing aids can directly receive the desired signal - without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

Whether a telecoil is built into your hearing aid is listed in section "Settings." If a telecoil is built in, you have the following options.

Ask your Hearing Care Specialist to configure a telecoil program.

- ▶ Switch to the telecoil program whenever you are in a place with an audio induction loop.

If a telecoil program is configured for your hearing aids, it is listed in section "Settings."

## Audio input (FM system)

With an audio input you can connect your hearing aids to an FM system. An FM system consists of a transmitter and a receiver. It gathers speech at the voice source (via a separate microphone) and transmits the sound directly to the hearing aids.

The signal is clearer and unaffected by poor room acoustics. This can be useful for adults in difficult listening situations and for children in classrooms.

Most FM systems allow for coupling with audio devices like an MP3 player.

### Notes:

- The battery compartment with audio input is available as an accessory only for certain aid types. Consult your Hearing Care Specialist for further information.
- Ask your Hearing Care Specialist to configure your hearing aids, for example:
  - Adjust the mix of the audio input signal with surrounding sounds.
  - Configure the controls so that you can select between audio input signal only, surrounding sound only, or a mix of both signals.

### **WARNING**

Risk of electric shock!

- ▶ Connect audio input only to a device conforming to IEC 60065, the IEC-Norm for audio, video and other electronic devices.

# Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

## Hearing aids

### Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ Ask your Hearing Care Specialist for recommended drying products.
- ▶ For longer periods of non-use, store your hearing aids with open battery compartment and batteries removed in a drying system to avoid the adverse effects of moisture.

### Cleaning

**Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.**

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

## Ear pieces and tubes

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality.

For hearing aids with a standard tube (ThinTube):

Clean the ear pieces daily, clean the tubes as required.

Exchange the ear pieces and tubes approximately every three to six months, or sooner if you notice cracks or other changes.

- For hearing aids with an earhook and standard ear piece:

Clean the ear pieces daily.

Exchange the ear pieces approximately every three to six months, or sooner if you notice cracks or other changes.

Ask your Hearing Care Specialist to clean or exchange the tubes as needed.

- For hearing aids with an earhook and custom-made ear piece:


Clean the ear pieces daily.

Ask your Hearing Care Specialist to clean the tubes or to exchange the ear pieces and tubes as needed.

## Cleaning ear pieces

- ▶ Clean the ear piece upon removal with a soft and dry tissue.

This prevents cerumen becoming dry and hardening.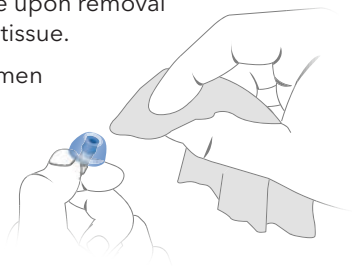


- ▶ Never use running water or immerse the ear piece in water.

## Cleaning standard tubes

Never use running water or immerse the tube in water.

You can clean standard tubes (ThinTube) using a special cleaning wire. Ask your Hearing Care Specialist for suitable cleaning wires.

- ▶ Remove tube and adapter by unscrewing the adapter from the hearing aid.
- ▶ Carefully insert the cleaning wire into the adapter ①.
- ▶ Push the cleaning wire all the way through the tube ②.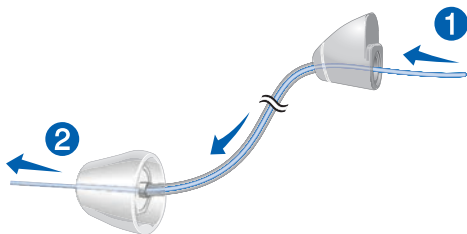

- ▶ Wipe away any cerumen or debris that is dislodged.
- ▶ Completely remove the cleaning wire.
- ▶ Re-attach tube and adapter to the hearing aid.


## Exchanging standard ear pieces

- ▶ Pull off the used ear piece and plug on the new ear piece.

with standard tube


with earhook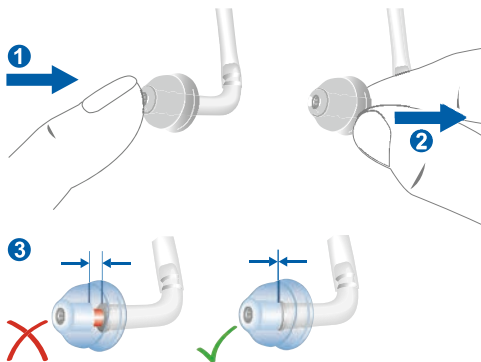

## Exchanging standard tubes and adapters

- ▶ Unscrew the used tube from the adapter ①.
- ▶ If necessary, unscrew the used adapter ②.

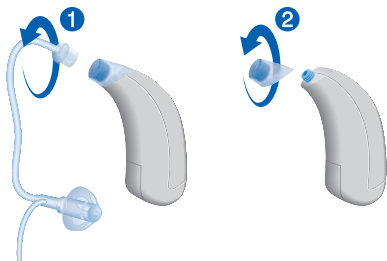- ▶ Screw on the new adapter and tube.

## Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Specialist.

Ask your Hearing Care Specialist for your individual recommendation of maintenance intervals and for support.

# Further information

## Safety information

For more safety information, refer to the safety manual supplied with the device.

## Symbols used in this document

---

Points out a situation that could lead to serious, moderate, or minor injuries.

---

Indicates possible property damage.

---

Advice and tips on how to handle your device better.

---

## Troubleshooting

### Problems and possible solutions

---

Sound is weak.

---

- Increase the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
-

## Problems and possible solutions

---

Hearing aid emits whistling sounds.

---

- Reinsert the ear piece until it fits securely.
  - Check whether tube, adapter and ear piece are connected correctly.
  - Decrease the volume.
  - Clean or exchange the ear piece.
- 

Sound is distorted.

---

- Decrease the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
- 

Hearing aid emits signal tones.

---

- Replace the empty battery.
- 

Hearing aid does not work.

---

- Turn the hearing aid on.
  - Gently close battery compartment completely.
  - Replace the empty battery.
  - Make sure the battery is inserted correctly.
  - Power-on delay is active.  
Wait several seconds and check again.
- 

Consult your Hearing Care Specialist if you encounter further problems.

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Derechos de autor ©2020 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.