MEENERGY™ RIC R

Receiver-In-The-Canal User Manual

Content

Welcome	4
<hr/>	
Your hearing aids	5
<hr/>	
Hearing aid type	5
Getting to know your hearing aids	6
Components and names	6
Controls	8
Hearing programs	9
Features	9
Daily use	10
<hr/>	
Charging	10
Turning on and off	11
Switching to standby mode	12
Inserting and removing your hearing aids	13
Adjusting the volume	17
Changing your hearing programs	17
Further adjustments (optional)	18
Special listening situations	19
<hr/>	
On the phone	19
Audio streaming with iPhone	20
Traveling on an airplane	20

Maintenance and care	21
Hearing aids	21
Ear pieces	23
Professional maintenance	25
Important safety information	26
Further information	28
Safety information	28
Accessories	28
Operating, transport, and storage conditions	29
Disposal information	30
Symbols used in this document	31
Troubleshooting	32
Country-specific information	33
Service and warranty	34

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Before using your hearing aids for the first time, fully charge them. Follow the instructions in the charger's user manual.

Your hearing aids

This user manual describes optional features that your hearing aids may have.

Ask your Hearing Care Specialist to indicate the features that apply to your hearing aids.

Hearing aid type

Your hearing aids are a RIC (Receiver-in-Canal) model. The receiver is placed within the ear canal and connected to the hearing aid via a receiver cable. The hearing aids are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

A power cell (lithium-ion rechargeable battery) is permanently built into your hearing aid. This enables easy charging with our inductive charger.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature Bluetooth® low energy* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone.**

*The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.


**iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

Getting to know your hearing aids

It is recommended you familiarize yourself with your new hearing aids. With the hearing aids in your hand, try using the controls and note their location on the hearing aid. This will make it easier for you to feel and press the controls while wearing the hearing aids.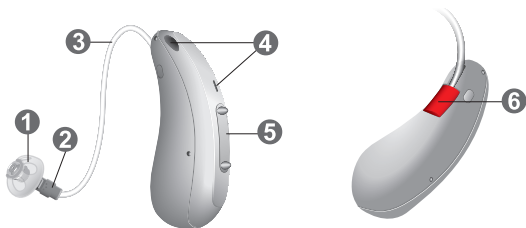

i If you have problems pressing the controls of your hearing aids while wearing them, you can ask your Hearing Care Specialist whether a remote control or a smartphone app to control your hearing aids is available.

Components and names

-
- | | |
|-----------------------|--|
| 1 Ear piece | 5 Rocker switch
(control, on/off switch) |
| 2 Receiver | |
| 3 Receiver cable | 6 Side indicator
(red = right ear,
blue = left ear) and
receiver connection |
| 4 Microphone openings | |
-

You can use the following standard ear pieces:

Standard ear pieces	Size
<input type="radio"/>  Click Sleeve (vented or closed)	
<input type="radio"/>  Click Dome™ single (open or closed)	
<input type="radio"/>  Click Dome semi-open	
<input type="radio"/>  Click Dome double	

You can easily exchange the standard ear pieces. Read more in section “Maintenance and care”.

Custom-made ear pieces

<input type="radio"/>  Custom shell	
<input type="radio"/>  Click Mold™	

Controls

The rocker switch allows you to switch hearing programs. Your Hearing Care Specialist has programmed your desired functions to the rocker switch.

Rocker switch function	L	R
Press briefly:		
Program up	<input type="radio"/>	<input type="radio"/>
Volume up	<input type="radio"/>	<input type="radio"/>
Volume down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
Press for about 2 seconds:		
Program up	<input type="radio"/>	<input type="radio"/>
Volume up	<input type="radio"/>	<input type="radio"/>
Volume down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
Press for more than 3 seconds:		
Turn on/off	<input type="radio"/>	<input type="radio"/>

L = Left, R = Right

You can also use a remote control to change hearing programs and adjust the volume of your hearing aids. With our smartphone app you have even more control options.

Hearing programs

1

2

3

4

5

6

Read more in section "Changing hearing programs."

Features

The **Tinnitus control feature** generates a sound to distract you from your tinnitus.

Daily use

To ensure easy use of your hearing aids, they are equipped with a rocker switch. In addition, we offer an app for Android® as well as iPhones® which makes handling your devices even easier. Contact your Hearing Care Specialist for help downloading and installing the smartphone app.

Charging

Place your hearing aids in the charger.

- ▶ Follow the instructions in the charger's user manual. You can also find useful tips for charging in the charger's user manual.

Low power indication

You will hear an alert signal to indicate the power cell is running low. The signal will be repeated every 20 minutes.

Depending on the usage of your hearing aids, you have approximately 1.5 hours to charge your hearing aids before they stop working.

Note that the power cell runtime is reduced if too many features are used simultaneously.

Turning on and off

You have the following options to turn your hearing aids on or off.

Via charger:

- ▶ Turning on: Take the hearing aids out of the charger.

The startup melody is played in your hearing aids.

The **default** volume and hearing program are set.

- ▶ Turning off: Place the hearing aids in the charger.

Note that the charger must be connected to a power supply. For details refer to the charger's user manual.

Via rocker switch:

- ▶ Turning on: Press and hold the lower part of the rocker switch until the startup melody starts to play. Release the rocker switch while the melody is playing.

The **default** volume and hearing program are set.

- ▶ Turning off: Press and hold upper or lower part of the rocker switch for several seconds. A power down melody is played.

See also the tips for charging in the charger's user manual.

Switching to standby mode

Via remote control or smartphone app, you can switch your hearing aids to standby mode. This mutes the hearing aids. When you leave standby mode, the **previously** used volume and hearing program are set.

Note:

- In standby mode, the hearing aids are **not** turned off completely. They draw some power.


Therefore, we recommend using standby mode only for a short period of time.

- If you want to leave standby mode, but the remote control or app is not available, turn your hearing aids off and on again (via rocker switch or by shortly placing them in the charger until one or more orange LEDs light up). In this case, the **default** volume and hearing program are set.

Inserting and removing your hearing aids

Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:


- **red marker** = right ear
- **blue marker** = left ear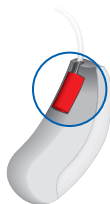


Inserting a hearing aid:

- ▶ For Click Sleeves, take care that the bend of the Click Sleeve is in line with the bend of the receiver cable.


Correct: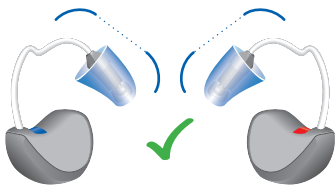

Incorrect:

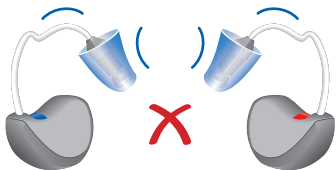- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal ①.
- ▶ Twist it slightly until it sits well.
Open and close your mouth to avoid accumulation of air in the ear canal.
- ▶ Lift the hearing hearing aid and slide it over the top of your ear ②.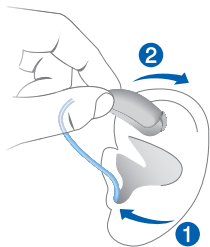

CAUTION

Risk of injury!


- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the **right** hearing aid with the **right** hand and the **left** hearing aid with the **left** hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).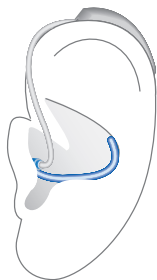

Removing a hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ①.
- ▶ If your hearing aid is equipped with a custom shell or a Click Mold, remove it by pulling the small removal cord toward the back of your head.

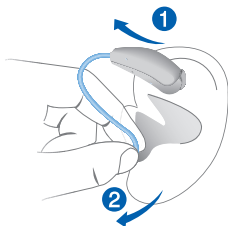- ▶ For all other ear pieces: Grip the receiver in the ear canal with two fingers and pull it carefully out ②.

Do not pull the receiver cable.

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section “Maintenance and care”.

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the upper part of the rocker switch to increase the volume or press the lower part to decrease the volume.

Refer to section "Controls" for the setting of your rocker switch.

An optional signal can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

Changing hearing programs

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the rocker switch briefly.

Refer to section "Controls" for the setting of the rocker switch. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

Further adjustments (optional)


You can also use the hearing aid controls to change other features, for example, the level of the Tinnitus control signal.

Refer to section "Controls" for the setting of your controls.

Special listening situations

On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver should be aligned. Turn the receiver slightly so that the ear is not completely covered.

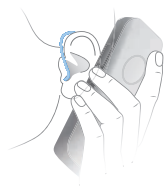Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs".

Audio streaming with iPhone®

Your hearing aids are **Made for iPhone** hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Specialist.

Traveling on an airplane

The charger is allowed in both, carry-on luggage and checked luggage.

You need to turn off the Bluetooth® connection on your hearing aids by using the app. There is no risk of transmitting from the hearing aids 2.4 GHz transmitter as long as the hearing aids are not connected to the mobile phone.

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids

Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

Drying

Your charger dries the hearing aids while charging. You can also use conventional products to dry your hearing aids. Ask your Hearing Care Specialist for recommended drying products and for individual instructions on when to dry your hearing aids.

Storage

- Short-term storage (up to several days): Turn off your hearing aids via the rocker switch or place them in the powered charger.

The charger must be connected to a power supply. If the charger is not connected to a power supply and you place your hearing aids in the charger, the hearing aids are **not** turned off.

Note that when you turn off the hearing aids via a remote control or smartphone app, the hearing aids are **not** turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing aids. Turn them off via the rocker switch before storing them. We recommend using drying products while storing your hearing aids.

At 6 month intervals you will have to charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged and have to be replaced. We recommend recharging more frequently than 6 month intervals.

Observe the storage conditions in section "Operating, transport and storage conditions".

Ear pieces


Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

For all types of ear pieces:


- ▶ Clean the ear piece right after removal with a soft and dry tissue.

This prevents that cerumen becomes dry and hard.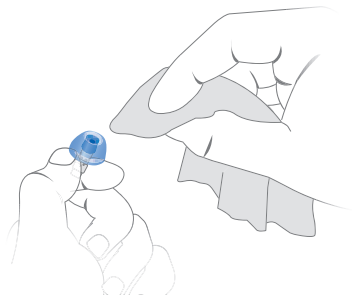

For Click Domes or Click Sleeves:

- ▶ Squeeze the tip of the ear piece.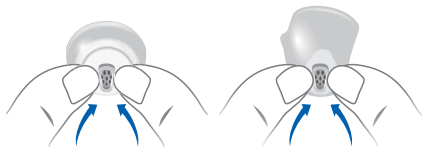


Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section “Components and names”, your Hearing Care Specialist has marked your type of ear piece.

Exchanging Click Domes

- ▶ To remove the old Click Dome, use the removal tool or follow the instructions on the packaging of the Click Domes. Be careful not to pull the receiver cable.

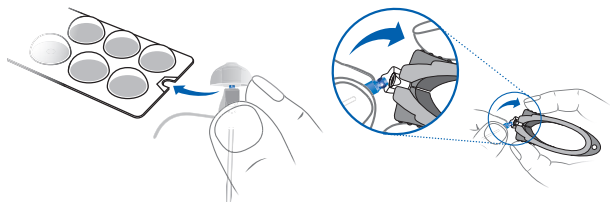The removal tool is available as an accessory.

- ▶ Take particular care that the new Click Dome clicks noticeably in place.
- ▶ Check the firm fit of the new Click Dome.

Exchanging Click Sleeves

- ▶ Turn the Click Sleeve inside out.
- ▶ Grasp the Click Sleeve and the receiver (not the receiver cable) with your fingers and pull the Click Sleeve off.
- ▶ Take particular care that the new Click Sleeve clicks noticeably in place.

Professional maintenance

Your Hearing Care Specialist can perform thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Specialist.

Ask your Hearing Care Specialist for recommended maintenance intervals and support.

Important safety information

This section contains important safety information regarding the power cell. For more safety information, refer to the safety manual supplied with the hearing aids.

WARNING

A power cell (lithium-ion rechargeable battery) is permanently built into your hearing aid. Incorrect use of the power cell or the hearing aid can cause the power cell to burst.

Risk of injury, chemical hazard, fire or explosion!

- ▶ Observe the operating conditions. Refer to section "Operating, transport and storage conditions".
- ▶ Pay special attention to protect your hearing aids from extreme heat (e.g. fire, microwave oven, inductive oven or other high-induction fields) and sunlight. Do not crush, short-circuit, incinerate or heat power cells or the hearing aid above 100 °C (212 °F).
- ▶ Charge your hearing aids only with the approved charger. Ask your Hearing Care Specialist for support.
- ▶ Do not disassemble the power cell.

WARNING

Incorrect use of the power cell or the hearing aid can cause the power cell to burst.

Risk of injury, chemical hazard, fire or explosion!

Replacement and repair have to be performed by a Service Center only. Therefore:

- ▶ Do not repair or replace the power cell yourself.
- ▶ Do not use the hearing aids when their performance after charging reduces considerably.
- ▶ In the unlikely event of a power cell leaking, bursting or exploding:
 - ▷ Keep away from all flammable materials.
 - ▷ If the electrolyte comes into contact with the skin, eyes or other parts of body, flush immediately with water. Consult a physician for medical advice.

In any of the above cases, return your hearing aids to your Hearing Care Specialist.

Further information

Safety information

For more safety information, refer to the safety manual supplied with the device.

Accessories

You can use the smartphone app for convenient remote control of your hearing aids. The app also allows you to deactivate signal tones, to create individual sound profiles and many more functions.

You can also use a remote control.

Consult your Hearing Care Specialist for further information.

Operating, transport, and storage conditions

Your hearing aids operate in the following environmental conditions (also valid in between uses):

Operating conditions	Discharging	Charging
Temperature	0 to 45 °C (32 to 113 °F)	10 to 30 °C (50 to 86 °F)
Relative humidity	5 to 93 %	5 to 93 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature (recommended)	10 to 25 °C (50 to 77 °F)	-20 to 60 °C (-4 to 140 °F)
Temperature (maximum range)	0 to 40 °C (32 to 104 °F)	-20 to 60 °C (-4 to 140 °F)
Relative humidity (recommended)	20 to 80 %	5 to 90 %
Relative humidity (maximum range)	10 to 80 %	5 to 90 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

Note that you have to charge the hearing aids at least every 6 months. We recommend to charge the hearing aids every 3 months. For details refer to section “Maintenance and care”.

For other parts, such as the charger, other conditions may apply.

WARNING

Risk of injury, fire or explosion during shipment.

- ▶ Ship power cells or hearing aids according to local regulations.

Disposal information

- ▶ To avoid environmental pollution, do not throw power cells or hearing aids into household trash.
- ▶ Recycle or dispose of power cells or hearing aids according to local regulations.

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Made for

 iPhone | iPad | iPod

“Made for iPhone”, “Made for iPad”, and “Made for iPod” mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits whistling sounds.

- Reinsert the ear piece until it fits securely.
 - Decrease the volume.
 - Clean or exchange the ear piece.
-

Sound is distorted.

- Decrease the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits signal tones.

- Charge the hearing aid.
-

Hearing aid does not work.

- Turn the hearing aid on.
 - Charge the hearing aid. Observe also the status LEDs on the charger.
-

Consult your Hearing Care Specialist if you encounter further problems.

Country-specific information

USA

Certification information for hearing aids with wireless functionality:

MEENERGY™ RIC R XP

- HVIN: RFM002
- Contains FCC ID: SGI-RFM002

Service and warranty

Serial numbers	Hearing Aid	Receiver unit
----------------	-------------	---------------

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Copyright ©2020 Miracle-Ear, Inc.
Distributed By: Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.