MEENERGY™ iRIC R & ME CHARGE™

User Manual

www.miracle-ear.com

Made for

 iPhone | iPad | iPod

Contents

Welcome	6
Intended use	8
Your hearing aids	9
Hearing aid type	9
Getting to know your hearing aids	11
Components and names	12
Hearing programs	15
Features	15
Daily use	16
Charging	16
Turning your hearing aids on and off	24
Switching to standby mode	25
Inserting and removing the hearing aids	26

Adjusting the volume	33
Changing the hearing program	34
Charging the internal battery of your charger	35
Checking the battery capacity of your charger	40

Special listening situations **42**

On the phone	42
Audio streaming with iPhone	44
Airplane mode for hearing aids	45

Frequently asked questions **46**

Maintenance and care **49**

Hearing aids	49
Ear pieces	52
Charger	57
Professional maintenance	58

Important safety information for hearing aids and charger 59

Safety information on lithium-ion rechargeable batteries 59

Safety information for hearing aids 62

Safety information for charger 62

Further information 69

Accessories 69

Operating, transport and storage conditions for
hearing aids and charger 70

Disposal information 72

Symbols 73

Troubleshooting for hearing aids 78

Troubleshooting for the charger 80

Country-specific information 83

Important information 84

Conformance information for charger 84

Service and warranty 85

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Before using your hearing aids for the first time, fully charge them.

Intended use

Hearing aids are intended to improve the hearing of hearing impaired persons. Diagnosis and prescription of a hearing aid must be performed by hearing health specialists, e.g. acousticians, audiologists or ENT doctors.

The charger is intended to charge Miracle-Ear® specific hearing aids with built-in power cells (lithium-ion rechargeable batteries).

Use the hearing aids and accessories only as described in the respective user guides.

Your hearing aids

This user guide describes optional features that your hearing aids may or may not have.

Ask your Hearing Care Specialist to specify the features that are valid for your hearing aids.

Hearing aid type

Your hearing aids are a RIC (Receiver-in-Canal) model. The receiver is placed within the ear canal and connected to the hearing aid via a receiver cable. The instruments are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

A power cell (lithium-ion rechargeable battery) is permanently built into your hearing aid. This enables easy charging with our charger.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature Bluetooth® low energy* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone**.

* The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

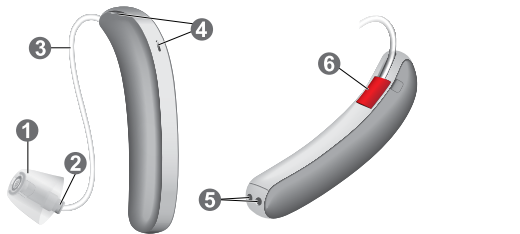
**iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

Getting to know your hearing aids

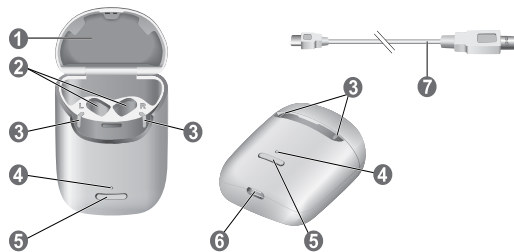
Your hearing aids have no controls.

If you want to adjust the volume or switch hearing programs manually, you can ask your Hearing Care Specialist about a remote control option.

Components and names

-
- | | |
|------------------|--|
| ① Ear piece | ④ Microphone openings |
| ② Receiver | ⑤ Charging contacts |
| ③ Receiver cable | ⑥ Side indicator
(red = right ear, blue = left ear) |
-

-
- | | |
|----------------------------|-----------------------------|
| ① Lid | ④ Status LEDs for Charger |
| ② Charging slots | ⑤ Button |
| ③ Status LEDs hearing aids | ⑥ USB port for power supply |
| | ⑦ USB power cable |
-

You can use the following standard ear pieces:

Standard ear pieces	Size
<input type="radio"/>  Click Sleeve (vented or closed)	
<input type="radio"/>  Click Dome™ single (open or closed)	
<input type="radio"/>  Click Dome semi-open	
<input type="radio"/>  Click Dome double	

You can easily exchange the standard ear pieces. Read more in section "Maintenance and care."

Hearing programs

1
2
3
4
5
6

Read more in section "Changing the hearing program."

Features

- The **tinnitus control feature** generates a sound to distract you from your tinnitus.

Daily use

Charging

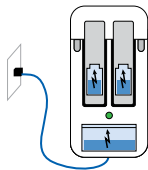
How you can use the charger

You can use your charger in three different ways.

Charging via electrical outlet

When the charger is plugged in or placed on an optional wireless Qi-charger, it charges the hearing aids (if they are placed in the charger) and it charges the integrated battery at the same time.

As long as the charger is being charged by an external power source, charger LED shows the charging status of the integrated battery and the LEDs for the hearing aids show the


status of the hearing aids.

For power saving after wireless charging, the LEDs will turn off once charger and hearing aids are fully charged.

On the go charging

When you are on the go with no electrical outlet available, you can still charge your hearing aids. Energy is fed from the charger's integrated battery into your hearing aids. The charger must be turned on to do so.

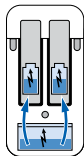
The charging status of the charger and hearing aids are shown by pushing the button or opening the lid.

Safekeeping

When the charging of hearing aids is completed, safely store your hearing aids inside the charger with the lid closed.

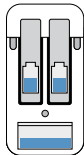If you want to store the hearing aids for a longer time, you can turn off the charger. The charger can only be turned off when there is no power source connected into the charger.

- When the charger turns on, the status of the hearing aids and the status of the charger's internal battery are displayed for 7 seconds.
- When the charger turns off, the front LED lights up green and fades out over a period of approximately 7 seconds. Shut down is complete after approximately 7 seconds.

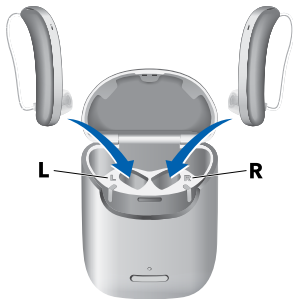
Turning the charger on or off

When the charger is connected to an electrical outlet, it is automatically turned on and cannot be turned off.

When charging on the go, press and hold the button for 5 seconds to turn the charger on or off.

Charging hearing aids

- ▶ Place the right hearing aid in the charging slot marked with an "R", and the left hearing aid in the charging slot marked with an "L."
- ▶ Stow the ear pieces gently.
- ▶ Close the charger's lid to save battery power.
- ▶ The hearing aids are automatically turned off and start charging. The charging status is shown for 7 seconds.

If the status LEDs do not light up when you insert the hearing aids, check the following:

- The hearing aids are not correctly placed. Push them slightly into their charging slots or close the charger's lid.
- The charger is turned off. Press and hold the button for 5 seconds to turn on the charger.
- The charger's internal battery is too low for on the go charging. Connect the charger to an electrical outlet.

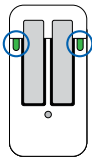
NOTICE

If you put a hearing aid in the wrong charging slot, the receiver cable may be damaged when closing the charger's lid.

Charging status of your hearing aids

The **LED** next to the hearing aid indicates the charging status of the hearing aid. Note that the status is only displayed immediately after putting the hearing aids in the charger, when you briefly press the button or when you open the lid. When the charger is off, no status is displayed at all.

LED Status description

Pulsing green
Hearing aid is being charged.

Green
Hearing aid is fully charged.

LED Status description

LED is off
No hearing aid is in the charger, hearing aid is not placed correctly, or charger is off.

Pulsing orange
Cooling phase. Charging is resumed when the device has cooled down. Please refer to section "Troubleshooting."

Blinking red
Hearing aid error. Please refer to section "Troubleshooting."

Low power indication of the hearing aids

You will hear an alert signal to indicate the power cell is running low. The signal will be repeated every 20 minutes. Depending on the usage of your hearing aids, you have approximately 1.5 hours to charge your hearing aids before they stop working.

Turning your hearing aids on and off

You have the following options to turn your hearing aids on or off.

- ▶ Turning on: Take the hearing aids out of the charger.
The startup melody is played in your hearing aids. The **default** volume and hearing program are set.
- ▶ Turning off: Place the hearing aids in the charger.

Note that the charger must be on.

Switching to standby mode

Via remote control, you can switch your hearing aids to standby mode. This will mute the hearing aids. When you leave standby mode, the **previously** used volume and hearing program are set.

Note:

- In standby mode, the hearing aids are not turned off completely. They draw some power.
Therefore, we recommend to use standby mode only for a short period of time.
- If you want to leave standby mode, but the remote control is not available: Turn your hearing aids off and on again (by shortly placing them in the charger until one or more LEDs light up). Wait until the startup melody is played in your hearing aids. This may take several seconds. Note that the **default** volume and hearing program are set.

Inserting and removing the hearing aids

Your hearing aids have been fine-tuned for your right and left ear.

Colored markers indicate the side:

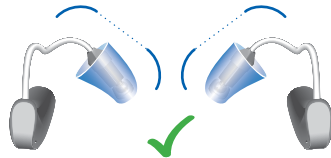
- Red marker = right ear
- Blue marker = left ear

Inserting a hearing aid:

- ▶ For Click Sleeves, take care that the bend of the Click Sleeve is in line with the bend of the receiver cable.


Correct:

Incorrect:

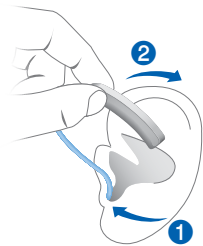- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal ①.
- ▶ Twist it slightly until it sits well.
Open and close your mouth to avoid accumulation of air in the ear canal.
- ▶ Lift the hearing aid and slide it over the top of your ear ②.

CAUTION

Risk of injury!

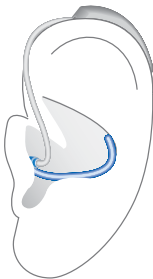
- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the right hearing aid with the right hand and the left hearing aid with the left hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

Removing a hearing aid:

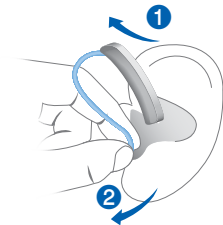- ▶ Lift the hearing aid and slide it over the top of your ear ①.

- ▶ Grip the receiver in the ear canal with two fingers and pull it carefully out ②.
Do not pull the receiver cable.

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care."

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, use a remote control option.

An optional signal can indicate the volume change.

Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program manually, use a remote control option.
Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

Charging the internal battery of your charger

NOTICE

Use of incompatible parts poses a risk of malfunction.

- ▶ Use only the USB power cable that is provided with the charger.
- ▶ Ensure your electrical plug complies with the requirements. Refer to the section "Requirements for the electrical plug."
- ▶ Connect the USB-C side of the USB power cable to the USB port of the charger ①.
- ▶ Connect the other side of the USB power cable to a suitable electrical plug ②.

The internal battery starts charging. When hearing aids are inserted, they start charging as well. The charging status of the charger is shown permanently.

Charging the internal battery with optional wireless Qi-charging

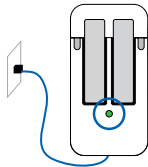
- ▶ Place the charger on a wireless Qi certified transmitter pad. After a few seconds the internal battery will start charging indicated by the pulsing green LED of the charger. When hearing aids are inserted they start charging as well indicated by the pulsing green LED of the hearing aids LED. If the LED is off, please re-align the charger to the center of the Qi transmitter pad.

The charging status of the charger is shown until it is fully charged. After that the LEDs are turned off for power saving reasons.

Charging status of your charger

The **front LED** indicates the charging status of the charger and its internal battery.

While the internal battery is being charged, the charging status is indicated as follows:

LED Status description

Pulsing green

Internal battery is being charged.

Green

Charger is fully charged.

LED is off (wireless charging)

Charger is fully charged.

Pulsing orange

Cooling phase. Charging is resumed when the device has cooled down. Please refer to section "Troubleshooting."

Blinking red

Charger error. Please refer to section "Troubleshooting."

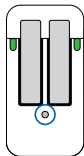
For an overview of the hearing aids' charging status, refer to the section "Charging hearing aids."

Checking the battery capacity of your charger

To check whether the charger's internal battery has enough capacity to fully charge your hearing aids when you are on the go:

- ▶ Disconnect the charger from the electrical outlet.
- ▶ Briefly press the button.

The **front LED** indicates the status of the charger's internal battery for 7 seconds.

LED Status description

Green

Internal battery has enough capacity to fully charge two hearing aids.

LED Status description

Orange

Internal battery does **not** have enough capacity to fully charge two hearing aids.

Blinking orange

Internal charger battery empty. Charging of hearing aids is not possible.

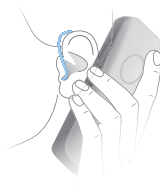
If the status LED does not light up when you press the button, check the following:

- The charger is off. Press and hold the button for 5 seconds to turn on the charger.
- The charger's internal battery is too low to display the status. Connect the charger to an electrical outlet, wait several minutes and try again.

Special listening situations

On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs."

Audio streaming with iPhone

Your hearing aids are **Made for iPhone** hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Specialist.

Airplane mode for hearing aids

Airplane mode is when the Bluetooth in your hearing aids is turned off. In areas where the use of Bluetooth technology is not allowed (e.g. on some airplanes), you can deactivate Bluetooth mode. This turns Bluetooth in your hearing aids off temporarily. The hearing aids still work without Bluetooth, however direct audio streaming will not be possible and other functions will not be available.

- ▶ To deactivate or activate the Bluetooth mode, use the smartphone app.

An alert tone indicates when Bluetooth mode is deactivated or activated.

Frequently asked questions

You may have some questions on how to best charge your hearing aids. Read the questions and answers below. If you have more questions, your Hearing Care Specialist will help you.

Why do my hearing aids not turn on when I take them out of the charger?

Check the following:

- The charger is off. Press and hold the button for 5 seconds to turn on the charger. Refer to the section "Turning the charger on or off."
- The charger's internal battery is too low. Connect the charger to an electrical outlet.

Now take your hearing aids out of the charger.

How can I turn off my hearing aids without charging them?

- ▶ Place the hearing aids in the charger. Ensure that they are inserted correctly (status LEDs light up). The hearing aids are automatically turned off and start charging.
- ▶ Disconnect the charger from the electrical outlet.
- ▶ Press and hold the button for 5 seconds to turn off the charger. Charging stops.

What should I pay attention to when charging?

The hearing aids and charger must be clean and dry. Observe the conditions in section "Operating, transport, and storage conditions." Avoid charging at high temperatures, as this can shorten the life span and capacity of the power cell.

If you use the charger in a very warm environment, note that it will stop charging if the hearing aids or the charger's temperature exceeds 42 °C (107 °F). The LED of the hearing aid and/or of the charger LED will pulsate orange during the cooling phase. Charging resumes automatically after the device has cooled down. Do not touch the hearing aids until the front LED is pulsing green, as the hearing aids may be hot.

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids

Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.

- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

Drying

Use conventional products to dry your hearing aids. Ask your Hearing Care Specialist for recommended drying products and for individual instructions on when to dry your hearing aids.

Storage

- Short-term storage (up to several days): Turn off your hearing aids by placing them in the charger.
The charger must be on. If the charger is not on and you place your hearing aids in the charger, the hearing aids are **not** turned off.

Note that when you turn off the hearing aids via a remote control or smartphone app, the hearing aids are **not** turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing aids. Leave them in the charger with the lid closed. Once the hearing aids are fully charged the charger and the hearing aids are turned off automatically.
We recommend using drying products while storing the hearing aids.
At 6 month intervals you will have to charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and they have to be replaced. We recommend recharging more frequently than 6 month intervals.
Observe the storage conditions in section "Operating, transport and storage conditions."

Ear pieces

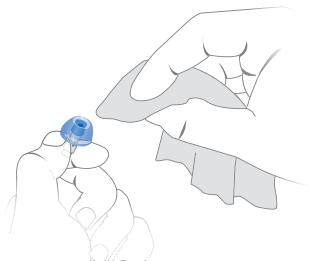
Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

For all types of ear pieces:


- Clean the ear piece right after removal with a soft and dry tissue.

This prevents that cerumen (ear wax) becomes dry and hard.

For Click Domes or Click Sleeves:

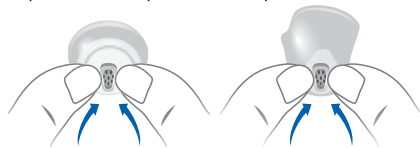- Squeeze the tip of the ear piece.


Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section "Components and names", your Hearing Care Specialist has marked your type of ear piece.

Exchanging Click Domes

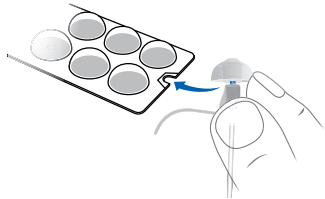- ▶ To remove the old Click Dome, follow the instructions on the packaging of the Click Domes. Be careful not to pull the receiver cable.


- ▶ Take particular care that the new Click Dome clicks noticeably in place.
- ▶ Check the firm fit of the new Click Dome.

Exchanging Click Sleeves

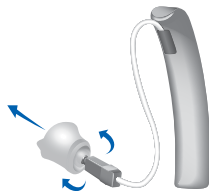- ▶ Turn the Click Sleeve inside out.
- ▶ Grasp the Click Sleeve and the receiver (not the receiver cable) with your fingers and pull the Click Sleeve off.
- ▶ Take particular care that the new Click Sleeve clicks noticeably in place.

Charger

Cleaning

NOTICE

Risk of injuries caused by electric shock!

- ▶ If cleaning is required, disconnect the charger from the power supply.
- ▶ Clean the charger with a soft, dry tissue.
- ▶ Never use running water or immerse the charger in water.
- ▶ Do not sterilize or disinfect the charger.

Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Ask your Hearing Care Specialist for recommended maintenance and support intervals for your individual needs.

Important safety information for hearing aids and charger

Safety information on lithium-ion rechargeable batteries

WARNING

A lithium-ion rechargeable battery is permanently built into your device. Incorrect use of the device can cause the lithium-ion rechargeable battery to burst.

Risk of injury, fire or explosion!

- ▶ Follow the safety instructions for lithium-ion rechargeable battery in this section.

- Observe the operating conditions. Refer to the section "Operating, transport, and storage conditions."
- Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- Charge your hearing aids only with the approved charger. Ask your Hearing Care Specialist for support.
- Do not disassemble the device.
- Do not use the device in explosive atmospheres.
- Do not use devices that are deformed or noticeably damaged.
- Do not use the device when its performance after charging decreases considerably. Return the device to your Hearing Care Specialist.
- Do not dispose of the device in fire or water.

Replacement and repair have to be performed by a professional. Therefore:

- Do not repair or replace the lithium-ion rechargeable battery yourself.
- In the unlikely event of a lithium-ion rechargeable battery bursting or exploding, keep all combustible materials away.

In any of the above cases, return your device to the Hearing Care Specialist.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC62368-1.

Safety information for hearing aids

For more safety information, refer to the safety manual supplied with the device.

Safety information for charger

Personal safety for charger

WARNING

Risk of electrical shock!

- ▶ Do not use noticeably damaged devices and return them to point of sale.

WARNING

Risk of injury!

- ▶ Do not use noticeably damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Specialist for support.

WARNING

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e. g. in mining areas).

**WARNING**

Risk of burns!

- ▶ Do not place anything other than your hearing aids in the charger.

**WARNING**

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and cognitively impaired persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

**WARNING**

Charging systems may interfere with measuring devices and electronic equipment.

- ▶ In hospitals: The charger complies with CISPR 35 and CISPR 32. Therefore the use of the charger in hospitals should be permissible. Nevertheless, the charger may interfere with highly sensitive devices. In those cases appropriate steps should be taken (increase distance from sensitive devices or disconnect the charger from power supply).

WARNING

Interference with implantable devices

- ▶ The device can be used with all electronic implants that comply with the ANSI/AAMI/ISO 14117:2012 Electromagnetic compatibility standard for human implants.
- ▶ In case your implant does not comply with the ANSI/AAMI/ISO 14117:2012 standard, consult the manufacturer of your implantable device for information about the risk of disturbance.

Product safety for charger

NOTICE

- ▶ Protect your device from extreme heat. Do not store it in direct sunlight.

NOTICE

- ▶ Protect your devices from high humidity.

NOTICE

- ▶ Do not dry your devices in the microwave oven.

NOTICE

- Do not drop. Internal components may get damaged.

Requirements for the electrical plug

Use an electrical plug with an output voltage of 5 V and a minimum output current of 500 mA.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC62368-1.

Further information

Accessories

The type of available accessories depends on the hearing aid type. Examples are: a remote control, an audio streaming device, or a smartphone app for remote control.

Consult your Hearing Care Specialist for further information.

Operating, transport and storage conditions for hearing aids and charger

Operating conditions	Discharging	Charging
Temperature	0 to 50 °C (32 to 122 °F)	10 to 30 °C (50 to 86 °F)
Relative humidity	5 to 93 %	20 to 75 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature (recommended)	10 to 25 °C (50 to 77 °F)	-20 to 60 °C (-4 to 140 °F)
Temperature (maximum range)	10 to 40 °C (50 to 104 °F)	-20 to 60 °C (-4 to 140 °F)
Relative humidity (recommended)	20 to 80 %	5 to 90 %
Relative humidity (maximum range)	10 to 80 %	5 to 90 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

Note that you have to charge the hearing aids at least every

6 months. For details refer to section "Maintenance and care."

WARNING

Risk of injury, fire or explosion during shipment.

- ▶ Ship power cells, hearing aids and/or charger according to local regulations.

Disposal information

- ▶ To avoid environmental pollution, do not throw power cells, hearing aids or charger into household trash.
- ▶ Recycle or dispose of power cells, hearing aids or charger according to local regulations.

Symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your hearing aids better.

Frequently asked questions.

Symbols on the device or packaging

EMC and radio communications compliance label
Australia, refer to section "Conformance information."

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic
waste. Read more in section "Disposal information."

Read and follow the instructions in the user guide.

According to IEC 60601-1, the applied parts of this
device are classified as type B.

Symbols on the device or packaging

Keep dry during transportation.

Temperature during transportation. Read more in
section "Operating, transport and storage conditions."

Relative humidity during transportation. Read more in
section "Operating, transport and storage conditions."

Atmospheric pressure during
transportation. Read more in section
"Operating, transport and storage conditions."

Medical device.

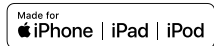
Symbols on the device or packaging

Compatible with Qi-certified wireless chargers.

Bluetooth® low energy technology.

“Made for iPhone”, “Made for iPad”, and “Made for iPod” mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

Troubleshooting for hearing aids

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits whistling sounds.

- Reinsert the ear piece until it fits securely.
 - Decrease the volume.
 - Clean or exchange the ear piece.
-

Problem and possible solutions

Sound is distorted.

- Decrease the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits signal tones.

- Charge the hearing aid.
-

Hearing aid does not work.

- Turn the hearing aid on.
 - Charge the hearing aid. Observe also the status LEDs on the charger.
-

Consult your Hearing Care Specialist if you encounter further problems.

Troubleshooting for the charger

Problem and possible solutions

No LED lights are on.

- Check if the power supply is connected correctly.
-

LEDs are pulsing orange.

Charger has reached the temperature limit and is now in a cooling phase. Charging is resumed after cooling down (about 15 minutes).

- Do not touch the hearing aids while the LEDs are flashing orange. The hearing aids may be hot.
 - If the charger enters cooling phase multiple times during charging, make sure to place the charger in an environment that complies with the recommended operating conditions.
-

Problem and possible solutions

Charger LED is blinking red.

An error in the charger has occurred.

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
- Reconnect the charger to the power supply and re-insert the hearing aids in the charging slots.

If the LEDs are still blinking red:

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
 - Contact your Hearing Care Specialist for support.
-

Problem and possible solutions

Hearing aid's LED is blinking red.

An error in the hearing aid has occurred.

- Take the respective hearing aid out of the charger and disconnect the charger from the power supply.
 - Contact your Hearing Care Specialist for support.
-

Consult your Hearing Care Specialist if you encounter further problems.

Country-specific information

USA

Certification information for hearing aids with wireless functionality:

MEENERGY iR XP:

- RF Module 2
- Contains FCC ID: SGI-RFM002

Important information

This device complies with Part 18 of the FCC Rules and with CAN RSS-216 / CNR-216.

Operation is subject to the following conditions:

- This device may not cause harmful interference
- This device must accept any interference received, including interference that may cause undesired operation
- The user shall keep a minimal distance of 20 cm (8 in) during the charging process

For conformance information for hearing aids, refer to the safety manual supplied with the device.

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Copyright ©2020 Miracle-Ear, Inc.
Distributed By: Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.