Miracle-Ear**LINK**TM Remote AV

User Manual

Introduction

Introduction

The 2.4 GHz Miracle-Ear**LINK**™ Remote AV allows for easy and discreet adjustments to your hearing aids.

With its long battery life, the Miracle-Ear**LINK**™ Remote AV is always ready to work with your hearing aids.

Introduction

- | | |
|-------------------------|------------------|
| 1. Ear Selection Switch | 5. Memory Change |
| 2. Volume Up | 6. Favorite |
| 3. Volume Down | 7. Home |
| 4. Mute/Unmute | 8. Lock/Unlock |

Pairing Process

For the Miracle-Ear**LINK**™ Remote AV to interact with your hearing aid(s), you must first pair the Miracle-Ear**LINK**™ Remote AV with each hearing aid. You only need to pair one time, as they should automatically recognize each other when both are powered on and within wireless operating range.

You can pair the Miracle-Ear**LINK**™ Remote AV with a single hearing aid or a set of hearing aids.

To start using the Miracle-Ear**LINK**™ Remote AV:

1. Move lower switch from Lock (right) to Unlock (left) position.

2. Power off, then power on your hearing aid(s), and place within 12 inches of the Miracle-Ear**LINK**™ Remote AV.

Note: For a CROS hearing aid system, power off both your hearing aid and CROS transmitter. Power on only your hearing aid, ensuring the CROS transmitter remains powered off.

3. Press and hold the HOME button for 5 seconds.

4. After 5 seconds, both LEDs start rapidly blinking.

5. For a **set of** hearing aids:

- If the left LED becomes solid blue for 2+ seconds and the right LED becomes solid red for 2+ seconds, then the pairing was successful. Otherwise, repeat steps 2 - 5.

For a **single left** hearing aid:

- If the left LED becomes solid blue for 2 seconds, then the pairing was successful. Otherwise, repeat steps 2 - 5.

For a **single right** hearing aid:

- If the right LED becomes solid red for 2 seconds, then the pairing was successful. Otherwise, repeat steps 2 - 5.

6. To verify your setup, pressing the Volume Up or Volume Down button should cause a hearing aid volume adjustment.

Basic Use

Ear Selection Switch

The Ear Selection Switch controls whether adjustments are made to one hearing aid or both hearing aids.

Set of Hearing Aids: To adjust both hearing aids at the same time, position the switch in the center.

Single Hearing Aid: The switch is ignored, as the Miracle-Ear**LINK**™ Remote AV always sends the appropriate left-side or right-side command.

LED

The blue LED blinks when a button press command is sent to the left hearing aid. Similarly, the red LED blinks for the right hearing aid.

Basic Use

Volume Up/Down

Short Press: The overall level of all sounds coming through your hearing aid microphone changes in the selected direction (either up or down).

Long Press: If streaming audio from a 2.4 GHz accessory, the audio stream volume changes in the selected direction (either up or down).

Home

To reset your hearing aids to memory one and the default volume level, press the Home button.

Lock/Unlock

To make adjustments to your hearing aids, the Miracle-Ear**LINK**™ Remote AV should be in the Unlock state. When not using your Miracle-Ear**LINK**™ Remote AV, you can lock it to prevent inadvertent hearing aid changes and to extend its battery life.

Basic Use

Mute/Unmute

To temporarily turn off sounds coming through your hearing aid microphone, press the Mute/Unmute button. To unmute the hearing aids, press the button again.

NOTE: When the hearing aids are muted, selecting any button will unmute the hearing aids as well as perform the function of the button selected.

Memory Change

Your Miracle-Ear® Hearing Care Professional may create multiple memories within your hearing aid. These additional memories can be accessed by pressing the Memory Change button, which will cycle through the available memories.

Basic Use

Favorite

Your hearing aids have the following Favorite button configuration:

- None** - Pressing this button has no effect on hearing aids.
- Stop/Start Audio Stream** - Pressing this button starts (or stops) audio streaming from a 2.4 GHz accessory.
- Comfort Boost** - Pressing this button reduces noise while optimizing sound quality in noisy environments. Press button again to deactivate this feature.
- Jump to Memory 4** - Pressing this button moves the hearing aid to memory four.
- Tinnitus Stimulus On/Off** - Pressing this button starts (or stops) playing a tinnitus stimulus through your hearing aids.


NOTE: Some of these features may not be available in all hearing aids.

Basic Use

Lanyard (optional)

1. Insert thin cord into end of the Miracle-EarLINK™ Remote AV housing until loop is exposed.
2. Thread lanyard through loop and pull lanyard.

NOTE: Lanyards (part #90974-000) can be obtained from your hearing professional.

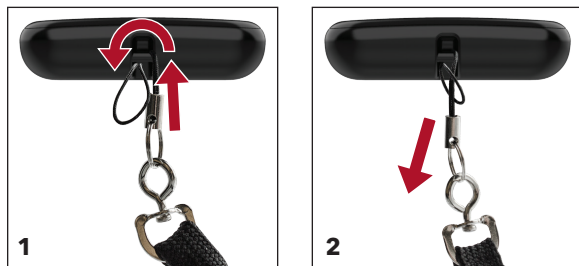Troubleshooting

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
LEDs do not illuminate when button is pressed.	The Miracle-EarLINK™ Remote AV is in locked state.	Move lower switch from Lock (right) to Unlock (left) position.
LEDs illuminate, but hearing aids do not respond to a button press.	After moving lower switch to Unlock position, it may take a brief time to wirelessly “locate” your hearing aids.	Wait for about a second after unlocking before pressing a button.
	Hearing aids are not paired with the Miracle-EarLINK™ Remote AV.	Perform pairing process with your hearing aids and Miracle-EarLINK™ Remote AV.
	Battery level in hearing aid can no longer support wireless activity.	Install new hearing aid battery.
Set of hearing aids fail to pair with the Miracle-EarLINK™ Remote AV.	Less than two hearing aids were found during pairing process.	During pairing process, make sure both hearing aids are available.
Only one hearing aid is receiving commands.	Ear selection switch is in the left or right position.	Move ear selection switch to center (both) position.

Regulatory Notices

Regulatory Notices

Required Information

Waste from electronic equipment must be handled according to local regulations.

Instructions for Disposal of Old Electronics

Miracle-Ear® encourages, and your local community laws may require, that your Miracle-Ear**LINK™** Remote AV be disposed of via your local electronics recycling/disposal process.

The below instructions are provided for the benefit of disposal/recycling personnel. Please include this manual when disposing of your Miracle-Ear**LINK™** Remote AV.

For Disposal/Recycling Personnel Only

This product contains a Lithium-ion Polymer battery. To remove this battery:

- Pry apart the housing halves with a small screwdriver.
- Unplug the battery connector from the circuit board to prevent accidental shorting.
- Pry the battery away from the housing using a wide flat blade.

Intended Use

The Miracle-Ear**LINK™** Remote AV is an accessory to a wireless air conduction hearing aid. It is intended to let a user remotely make volume and program adjustments to an air conduction hearing aid.

Regulatory Notices

The Miracle-Ear**LINK™** Remote AV is classified as a Type B applied part under the IEC 60601-1 medical instrument standard.

The Miracle-Ear**LINK™** Remote AV is designed to operate in public and residential environments and are designed to comply with international Electromagnetic Compatibility emissions and immunity standards for medical devices. However, it is still possible that you may experience interference caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

The Miracle-Ear**LINK™** Remote AV is designed to be cleaned with a damp cloth moistened with soapy water.

If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Miracle-Ear**LINK™** Remote AV might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance. The Miracle-Ear**LINK™** Remote AV should not be used during an MRI procedure or in a hyperbaric chamber, or other oxygen-rich environments. The Miracle-Ear**LINK™** Remote AV is not formally certified to operate in explosive atmospheres such as may be found in coal mines or certain chemical factories.

Use on Aircrafts

The Miracle-Ear**LINK™** Remote AV should not be used on aircrafts unless specifically permitted by the flight personnel.

Regulatory Notices


International Use

The Miracle-Ear**LINK™** Remote AV is approved to operate at radio frequencies that are specific to your country or region and might not be approved for use outside your country or region.

Be aware that operation during international travel may cause interference with other electronic devices, or other electronic devices may cause interference with your Miracle-Ear**LINK™** Remote AV.

The battery in your Miracle-Ear**LINK™** Remote AV is not replaceable. Do not attempt to replace the battery.

The Miracle-Ear**LINK™** Remote AV is designed to the following temperature ranges:

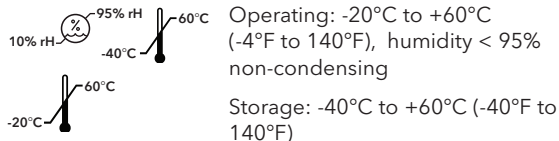We are required by regulations to provide the following warnings:

⚠ WARNING: Use of the Miracle-Ear**LINK™** Remote AV directly next to other electronic equipment should be avoided because it could result in improper performance. If such use is necessary, note as to whether your hearing aids and the other equipment are operating normally.

⚠ WARNING: Use of accessories, components or replacement parts other than those provided by the manufacturer of the Miracle-Ear**LINK™** Remote AV could result in increased electromagnetic emissions and decreased electromagnetic immunity and could result in degradation of performance.

Regulatory Notices

⚠ WARNING: If Portable Radio Frequency communications equipment is used closer than 30 cm (12 inches) from your Miracle-Ear**LINK™** Remote AV, degradation of the performance of your hearing aid could result. If this occurs, move away from the communications equipment.

Wireless Technical Description

The Miracle-Ear**LINK™** Remote AV operates in the 2.4 - 2.4835 GHz band with a maximum effected radiated power of 3 dBm with transmission modulation type of 1M06F1W. The receiver section of the radio has a bandwidth of 2 MHz.

The Miracle-Ear**LINK™** Remote AV has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m as well as proximity magnetic fields as defined in table 11 of 60601-1-2.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.

The Remote has a rating of IP 5X per IEC 60529. This means that the Remote is protected from dust.

The Remote does not contain any user-serviceable parts. Do not open or modify this device.

Do not connect the Remote to any external device.

The expected service life of the Remote is 3 years.

Regulatory Notices

FCC/IC Information

FCC ID: EOA-24WR

IC: 6903A-24WR (Model 610)

FCC/IC Notice

This device complies with part 15 of the FCC rules and with Industry Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions:

Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) this device must accept any interference received,

including interference that may cause undesired operation of the device.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Regulatory Notices

Manufactured For:

Miracle-Ear

150 South Fifth Street, Suite 2300

Minneapolis, MN 55402

The following symbols are used in this document and on the Remote:

Produces and receives radio frequency energy. Interference may occur in the vicinity of equipment marked with the following symbol.

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The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks is under license.
All trade names and trademarks are properties of their respective owners.
Distributed By Miracle-Ear, Inc.