## **MEMINI™ RIC 10**

with GENIUS™ 3.0 Technology User Manual

# Content

<b>Welcome</b>	<b>4</b>
<hr/>	
<b>Your hearing aids</b>	<b>5</b>
<hr/>	
Hearing aid type	5
Components and names	6
Controls	8
Hearing programs	9
Features	9
<b>Batteries</b>	<b>10</b>
<hr/>	
Battery size and handling tips	10
Replacing batteries	11
<b>Daily use</b>	<b>12</b>
<hr/>	
Turning on and off	12
Inserting and removing the hearing aids	13
Adjusting the volume	17
Changing the hearing program	17
<b>Special listening situations</b>	<b>18</b>
<hr/>	
On the phone	18
<b>Maintenance and care</b>	<b>19</b>
<hr/>	
Hearing aids	19
Ear pieces	20
Professional maintenance	22

<b>Further information</b>	<b>23</b>
Safety information	23
Accessories	23
Symbols used in this document	23
Troubleshooting	24
Service and warranty	25

# Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

## **CAUTION**

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

# Your hearing aids

This user guide describes optional features that your hearing aids may or may not have.


Ask your Hearing Care Specialist to indicate the features that are valid for your hearing aids.

## Hearing aid type

Your hearing aids are a RIC (Receiver-in-Canal) model. The receiver is placed within the ear canal and connected to the hearing aid via a receiver cable. The hearing aids are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.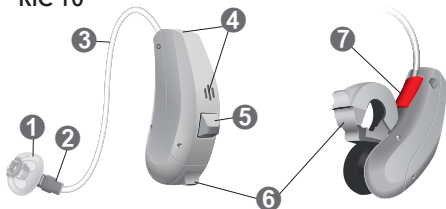

## Components and names

### MEMINI™ RIC 10

- 
- |                          |   |
|--------------------------|---|
| ① Ear piece              | ⑥ Battery compartment (on/off switch)                                       |
| ② Receiver               |   |
| ③ Receiver cable         | ⑦ Side indicator (red = right ear, blue = left ear) and receiver connection |
| ④ Microphone openings    |   |
| ⑤ Push button (optional) |   |
-

You can use the following standard ear pieces:

Standard ear pieces	Size
<input type="radio"/> 	Click Sleeve (vented or closed)
<input type="radio"/> 	Click Dome™ single (open or closed)
<input type="radio"/> 	Click Dome semi-open
<input type="radio"/> 	Click Dome double

You can easily exchange the standard ear pieces. Read more in section "Maintenance and care".

Custom-made ear pieces	
<input type="radio"/> 	Custom shell
<input type="radio"/> 	Click Mold™

## Controls

With the controls you can, for example, switch hearing programs. Your hearing aids either have a push button or no control.

Your Hearing Care Specialist has programmed your desired functions to the push button.

<b>Control</b>	<b>Left</b>	<b>Right</b>
Push button	<input type="radio"/>	<input type="radio"/>
No control	<input type="radio"/>	<input type="radio"/>

You can also ask your Hearing Care Specialist whether a remote control or a smartphone app to control your hearing aids is available.

<b>Push button function</b>	<b>L</b>	<b>R</b>
Press briefly:		
Program change	<input type="radio"/>	<input type="radio"/>
Press long:		
Standby/turn on	<input type="radio"/>	<input type="radio"/>

L = Left, R = Right

## Hearing programs

---

1

---

2

---

3

---

4

---

5

---

6

---

Read more in section "Changing the hearing program".

## Features

- Power-on delay** enables whistle-free insertion of hearing aids.  
Read more in section "Turning on and off".
- The tinnitus control** feature generates a sound to distract you from your tinnitus.

# Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

## Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.

---

<b>Battery size:</b>	<b>10</b>
----------------------	-----------

---

- Always use the correct battery size for your hearing aids.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

## Replacing batteries

### Removing the battery:

- ▶ Open the battery compartment.

- ▶ Use your finger or an optional magnet stick to remove the battery. The magnet stick is available as an accessory from your Hearing Care Specialist.

### Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.

- ▶ Insert the battery with the "+" symbol facing up (refer to the picture).

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly.

Do not attempt to close the battery compartment by force. It can be damaged.

# Daily use

## Turning on and off

You have the following options to turn your hearing aids on or off.

### Via battery compartment:

- ▶ Turning on: Close the battery compartment.  
The **default** volume and hearing program are set.
- ▶ Turning off: Open the battery compartment to the first catch.

### Via push button:

- ▶ Turning on or off: Press and hold the push button for several seconds. Refer to section "Controls" for the settings of your controls.

After turning on, the previously used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when a hearing aid is being turned on or off.


When the **power-on delay** is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

## Inserting and removing the hearing aids

Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:


- Red marker = right ear
- Blue marker = left ear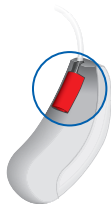


### Inserting a hearing aid:

- ▶ For Click Sleeves, make sure that the bend of the Click Sleeve is in line with the bend of the receiver cable.


Correct: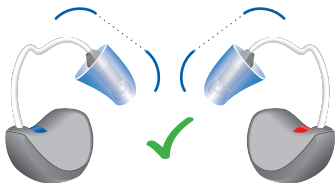

Incorrect:

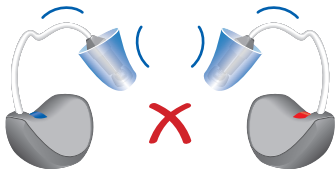- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal ①.
- ▶ Twist it slightly until it sits well.  
Open and close your mouth to avoid accumulation of air in the ear canal.
- ▶ Lift the hearing aid and slide it over the top of your ear ②.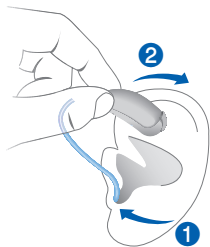

### CAUTION

Risk of injury!


- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the right hearing aid with the right hand and the left hearing aid with the left hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).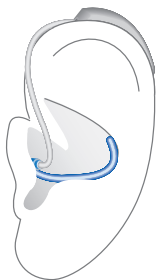

## Removing the hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ❶.
- ▶ If your hearing aid is equipped with a custom shell or a Click Mold, remove it by pulling the small removal cord toward the back of your head.

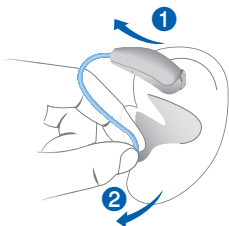- ▶ For all other ear pieces: Grip the receiver in the ear canal with two fingers and pull it carefully out ❷.

Do not pull the receiver cable.

### CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section “Maintenance and care”.

## Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, use a remote control option.

An optional signal can indicate the volume change.

## Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the push button briefly.

Refer to section "Controls" for the setting of your controls. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.


You can also ask your Hearing Care Specialist whether a remote control or a smartphone app to control your hearing aids is available.

## Special listening situations

### On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.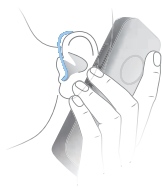

### Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section “Hearing programs”.

# Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

## Hearing aids

### Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ For longer periods of non-use, store your hearing aids in a dry environment with an open battery compartment and the batteries removed.
- ▶ Ask your Hearing Care Specialist for more information.

### Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.

- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

## Ear pieces


### Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

#### For all types of ear pieces:


- ▶ Clean the ear piece right after removal with a soft and dry tissue.

This prevents cerumen from becoming dry and hard.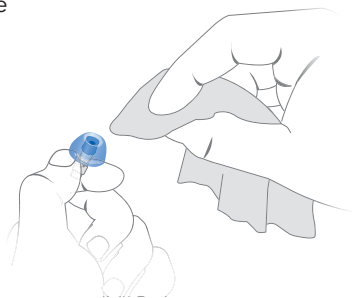

#### For Click Domes or Click Sleeves:

- ▶ Squeeze the tip of the ear piece.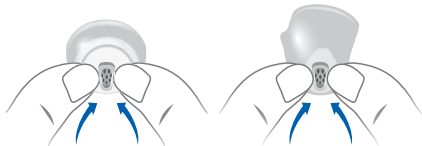


## Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section “Components and names,” your Hearing Care Specialist has marked your type of ear piece.

### Exchanging Click Domes

- ▶ To remove the old Click Dome, use the removal tool or follow the instructions on the packaging of the Click Domes. Be careful not to pull the receiver cable.

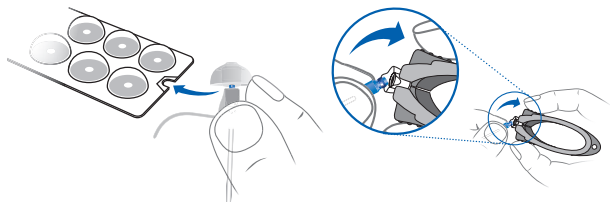The removal tool is available as an accessory.

- ▶ Make sure that the new Click Dome clicks noticeably in place.
- ▶ Check the firm fit of the new Click Dome.

## Exchanging Click Sleeves

- ▶ Turn the Click Sleeve inside out.
- ▶ Grasp the Click Sleeve and the receiver (not the receiver cable) with your fingers and pull the Click Sleeve off.
- ▶ Make sure that the new Click Sleeve clicks noticeably in place.

## Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Specialist.

Ask your Hearing Care Specialist for their recommendation of maintenance intervals and support.

# Further information

## Safety information

For more safety information, refer to the safety manual supplied with the device.

## Accessories

The type of available accessories depends on the hearing aid type. Examples are: a remote control or a smartphone app for remote control.

Consult your Hearing Care Specialist for further information.

## Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

---

## Troubleshooting

### Problem and possible solutions

---

Sound is weak.

---

- Increase the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
- 

Hearing aid emits whistling sounds.

---

- Reinsert the ear piece until it fits securely.
  - Decrease the volume.
  - Clean or exchange the ear piece.
- 

Sound is distorted.

---

- Decrease the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
- 

Hearing aid emits signal tones.

---

- Replace the empty battery.
- 

Hearing aid does not work.

---

- Turn the hearing aid on.
  - Gently close the battery compartment completely.
  - Replace the empty battery.
  - Make sure the battery is inserted correctly.
  - Power-on delay is active. Wait several seconds and check again.
- 

Consult your Hearing Care Specialist if you encounter further problems.

## Service and warranty

Serial numbers	Hearing aid	Receiver unit
----------------	-------------	---------------

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Copyright ©2020 Miracle-Ear, Inc. All rights reserved.
Distributed By Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.