ME STREAM

TV Streamer User Manual

www.miracle-ear.com

Content

Before you start **4**

Included in delivery 6

Components 8

Getting started **10**

Positioning 10

Connecting to power supply 11

Connecting to your electronic device 13

Pairing hearing aids 20

Daily use **23**

Using the transmitter 23

Power and TV indicator 26

Important safety information **30**

Personal safety 30

Product safety 33

Important information **36**

Explanation of symbols 36

Maintenance and care 39

Troubleshooting 41

Technical information 46

Disposal information 48

Conformance information 48

Before you start

This device is intended as a wireless transmitter of sound from a TV or from other electronic devices to hearing aids. It is intended to be used by children (> 36 months) and adults together with their hearing aids.

CAUTION

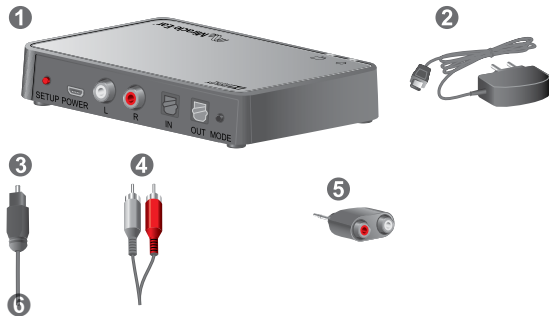
Read this user manual thoroughly and completely and follow the safety information in this document to avoid damage or injury.

Compatible hearing aids

The TV Streamer works with specific Miracle-Ear® wireless hearing aids. Your Hearing Care Specialist will advise you on the compatible models.

Included in delivery

- 1 TV Streamer
- 2 USB power cable and power plug with country-specific adapter
- 3 TOSLINK cable
- 4 RCA (cinch) cable
- 5 Mini jack adapter

Components

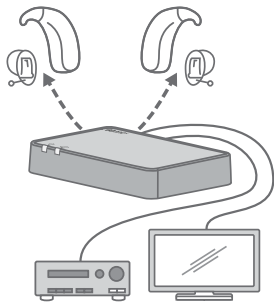
- ① TV indicator 
- ② Power indicator 
- ③ Setup button
- ④ Power connection
- ⑤ RCA connections (left and right)
- ⑥ TOSLINK connections (in and out)
- ⑦ Mode button (only for Hearing Care Specialists)

Getting started

Positioning

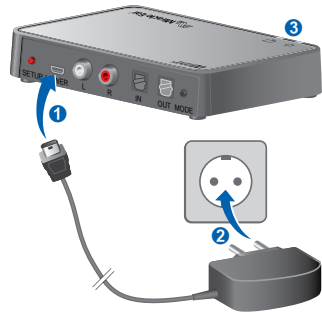
Position the transmitter close to your TV electronic device, so that it can be easily connected with cables. For best connection to your hearing aids, position the transmitter horizontally and in direct line of sight with your hearing aids.

Connecting to power supply

Only use the USB power cable and power plug that came with the transmitter.

- ▶ Connect the micro USB side of the power cable to the power connection of the transmitter **1**.
- ▶ Connect the power plug to the power supply **2**.

► Observe the indicators ③:

  Description

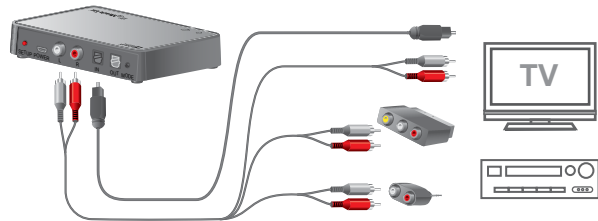
 - The transmitter is starting up.

  If after a few seconds, both indicators flash blue, the transmitter is on but not paired to any hearing aid. Refer to the section "Pairing hearing aids."

other Refer to the section "Power and TV indicator."

Connecting to your electronic device

You can connect the transmitter to your TV or electronic device with the TOSLINK cable or with the RCA cable. We recommend using the TOSLINK connection. Use the supplied cables and adapters.

We recommend connecting to either the TOSLINK input or the RCA input of the transmitter.

If both inputs of the transmitter are used, the electronic device that is connected via TOSLINK cable has the higher priority and only the audio signal of this device will be streamed to your hearing aids.

TOSLINK connection

- ▶ Before connecting the cable, remove the protection caps from the TOSLINK cable plugs.
- ▶ Plug the cable into the TOSLINK input of your transmitter and into the TOSLINK (optical audio) output of your electronic device.

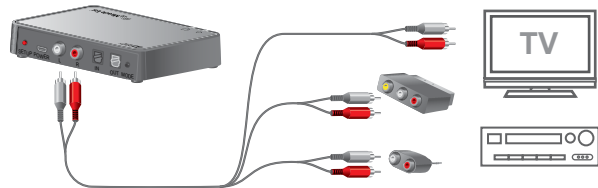
To prevent the TOSLINK cable from breaking, do not forcefully bend it.

Further information:

- Refer to the electronic device's manual: Set the output format of the electronic device to either PCM (pulse-code modulation) or Dolby® Digital.*
- The TOSLINK output of your transmitter may be used to connect further TOSLINK devices such as external Dolby decoders or surround sound systems.

* Dolby and the double-D symbol are registered trademarks of Dolby Laboratories.

RCA connection

- ▶ Plug the cable into the RCA connections of your transmitter.
- ▶ Plug the other side of the cable into the RCA output of the electronic device, or via adapter into the SCART or mini jack output of the electronic device.

Further information:

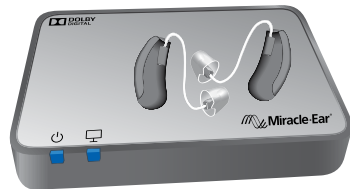
- Without adapter: The RCA output of the electronic device is typically marked "L-R AUDIO OUT" and found on the back or control panel.
- With SCART adapter: The audio signal can depend on the source selected on the electronic device. For example some TVs will only make TV sound available on the SCART output, but not the sound from a connected DVD player.
- With mini jack adapter: The mini jack output on the electronic device is typically the headphone output. The speakers of the electronic device may be muted if the headphone output is used so sound is only heard through the hearing aids.
- Refer to the electronic device's manual: Set the output format of the electronic device to analog audio or uncompressed audio.

Pairing hearing aids

Your hearing aids need to be paired once with the transmitter. After that, the pairing is saved in the hearing aids and remembered each time they are turned on.

- ▶ Ensure that the transmitter is connected to the power supply.
- ▶ Turn the hearing aids off and on again by opening and closing the battery compartments. If your hearing aids do not have a battery compartment, turn them off and on as described in the hearing aids' user manual.

- ▶ Place both hearing aids close to, or on top of the transmitter. The pairing process starts automatically.

- ▶ Observe the indicators:

		Description
		If pairing is completed, both indicators are blue for about 30 seconds.
other		Refer to the section "Power and TV indicator".

Multiple users:

- This transmitter can be paired to multiple hearing aids. All users will be able to hear the same sound.
- To clear the pairings of one user, first reset the transmitter to factory settings. This will clear the pairings of all users. After that, pair the hearing aids of the other users again. Refer to the section "Resetting to factory settings".

Daily use

Using the transmitter

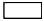
Function	Description
Start audio streaming	Switch your hearing aids to the transmitter mode or change to the dedicated hearing program.
Stop audio streaming	Quit transmitter mode or switch to any other hearing program in your hearing aids.
Adjust volume	Use the controls of your hearing aids.

Further information:

- Depending on your hearing aids' type and configuration, you can change their mode or hearing program with the hearing aids' controls (e. g. push button), or with a smartphone app, or with a dedicated remote control. Refer to the hearing aids' user manual for details.
- Volume functions and hearing programs are optional hearing aid features. They depend on the type of hearing aid and can be programmed by your Hearing Care Specialist.
- The volume of the streamed audio signal is normally not affected by volume adjustments from the TV remote control.
- Your Hearing Care Specialist can adjust the mix of the audio streaming signal with surrounding sounds according to your preference.

Power and TV indicator

Both power indicator  and TV indicator  show the current status of the transmitter.

		Description
	-	Power indicator is white: The transmitter is starting up.
		Both indicators flash blue: Power is on. The transmitter is not paired to any hearing instrument. Refer to the section "Pairing hearing aids."

		Description
		Both indicators are blue for 30 seconds: Pairing is complete.
	-	Power indicator is red: Power is on. The transmitter is not receiving an audio signal. Refer to the section "Troubleshooting".
		Both indicators are green: The transmitter is receiving an audio signal via the RCA connection.

  Description

  Power indicator is green, TV indicator flashes green:
The transmitter is not receiving an audio signal via the RCA connection or the volume is too low.

  Power indicator is green, TV indicator is blue:
The transmitter has a valid TOSLINK connection to your electronic device regardless of whether an audio signal is played or not.

  Description

  Power indicator is green, TV indicator flashes red:
The signal format is not supported. Refer to the section "Connecting to your electronic device".

  Both indicators flash red:
Error. Contact your Hearing Care Specialist.

Important safety information

Personal safety

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and cognitively impaired persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

Risk of electrical shock!

- ▶ Do not use obviously damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Specialist for support.

WARNING

Risk of electric shock!

- ▶ Connect audio input only to a device conforming to IEC 60065, the IEC-Norm for audio, video and other electronic devices.

WARNING

This device may interfere with measuring and electronic equipment.

- ▶ Do not use your device on airplanes or areas where sensitive or life-supporting electronic equipment is used.

WARNING

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e. g. in mining areas).

Product safety

NOTICE

- ▶ Protect your device from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Protect your device from high humidity.

NOTICE

- ▶ Do not dry your device in the microwave oven.

NOTICE

Different types of strong radiation, e. g. during X-ray or MRI head examinations, may damage devices.

- ▶ Do not use the device during these or similar procedures.

Weaker radiation, e. g. from radio equipment or airport security, does not damage the device.

In some countries restrictions for the usage of wireless equipment exist.

- ▶ Refer to local authorities for further information.

NOTICE

Your device is designed to comply with international standards on electromagnetic compatibility but interference with nearby electronic devices could occur. In this case, move away from the source of interference.

Important information

Explanation of symbols

Symbols used in this document

 Points out a situation that could lead to serious, moderate, or minor injuries.

 Indicates possible property damage.

 Advice and tips on how to handle your device better.

Symbols on the device or packaging

 CE compliance label, confirms compliance with certain European Directives, refer to section "Conformance information".

 EMC and radio communications compliance label Australia, refer to section "Conformance information".

 Indicates the legal manufacturer of the device.

Symbols on the device or packaging

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user manual.

Manufactured under licence from Dolby Laboratories.

Maintenance and care

NOTICE

- ▶ Do not put your device in water!
- ▶ Do not clean your device with alcohol or benzine.
- ▶ Clean the device as necessary with a soft tissue.
- ▶ Avoid abrasive cleaning liquids with additives such as citric acids, ammonia, etc.

Resetting to factory settings

To reset the transmitter to factory settings:

- ▶ Press and hold the setup button for more than 5 seconds.

All pairings are deleted. Both indicators flash blue.

Troubleshooting

If audio streaming is not working, check the following tables.

TOSLINK cable is connected

  Troubleshooting

  The signal format is not supported. Refer to the section "Connecting to your electronic device".

  No audio signal is available. For example, pause or mute is pressed on the electronic device.

  Your hearing aids are out of operating distance. Move closer to the transmitter.

  Troubleshooting

  Your hearing aids have not been paired with the transmitter, but other hearing aids are paired. Pair your hearing aids. Refer to the section "Pairing hearing aids".

 - No audio signal is available. For example, pause or mute is pressed on the electronic device.

 - TOSLINK cable is defective. Check the cable for damages and replace it if necessary.

RCA cable is connected

  Troubleshooting

  The output signal of the electronic device is not loud enough. Increase the volume on the electronic device.

  No audio signal is available. For example, pause or mute is pressed on the electronic device.

  Your hearing aids are out of operating distance. Move closer to the transmitter.

  Troubleshooting

-   Your hearing aids have not been paired with the transmitter, but other hearing aids are paired. Pair your hearing aids. Refer to the section "Pairing hearing aids".
 -   The signal format is not supported. Refer to the section "Connecting to your electronic device".
 -  - The output signal of the electronic device is not loud enough. Increase the volume on the electronic device.
-

  Troubleshooting

-  - No audio signal is available. For example, pause or mute is pressed on the electronic device.
-

Both TOSLINK and RCA cable are connected

-  Unplug the cable that is currently not in use (either TOSLINK or RCA cable).

Technical information

Operating, transport, and storage conditions

Operating conditions

Temperature	5 to 40 °C
Relative humidity	15 to 93 %, non-condensing
Atmospheric pressure	700 to 1060 hPa

During extended periods of transport and storage, please observe the following conditions:

Transport and Storage

Temperature	-25 to 70 °C
Relative humidity	15 to 93 %, non-condensing

For other parts, such as hearing aids, other conditions may apply.

Technical data of the power supply

Input voltage:	100-240 V AC
Input current:	max. 0.2 A at 100 V AC
Input frequency:	47-63 Hz
Output voltage:	5 V DC
Output current:	max. 1 A

Disposal information

Recycle devices, accessories and packaging according to local regulations.

Conformance information

The CE mark indicates conformity with the following European directives: 93/42/EEC concerning medical devices, 99/5/EC (R&TTE) concerning radio and telecommunications terminal equipment, and 2011/65/EU RoHS concerning the restriction of hazardous substances.

Wireless functionality

This device contains a wireless transmitter using 2.4 GHz technology. The emission power from this device is max. 17.1 dBm EIRP. This device complies with international standards concerning electromagnetic compatibility and human exposure.

FCC and ISED compliance information can be found on the back of the device.

The ACMA compliance mark  indicates conformity with the electromagnetic interference standards set by the Australian Communications and Media Authority (ACMA).

Devices with the FCC marking comply with the standards of the FCC regarding electromagnetic interference.

This Class B digital apparatus complies with Canadian ICES-003.

This device complies with Part 15 of the FCC Rules and with ISED's licence-exempt RSSs.

Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a

residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

To comply with FCC RF exposure requirements, the device and the antenna for this device must be installed to ensure a minimum separation distance of 20 cm or more from a person's body. Other operating configurations should be avoided.

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Derechos de autor ©2020 Miracle-Ear, Inc.
Distributed by Miracle-Ear, Inc.